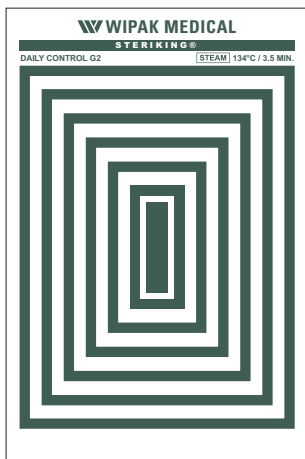
UK **Completely colour-changed sheet.**
Minimum uniform colour change that must be achieved.

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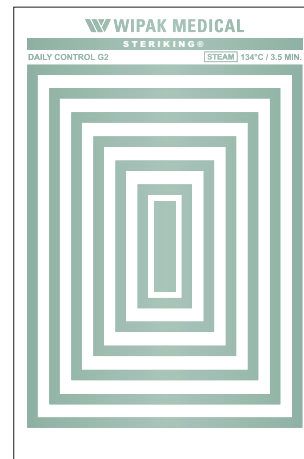
UK **Completely colour-changed sheet (3,5 min. 134°C).**
Maximum uniform colour change that can be achieved.

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UK **Not properly colour-changed sheet.**
Possible causes: sterilization time too short or temperature too low.

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Instructions for use

Conforming to EN-ISO 11140-1 (Class 2+6), EN-ISO 11140-4

Daily Control DCG2 is a Bowie & Dick type routine test for prevacuum autoclaves and should be used in the same manner as the standard Bowie & Dick test pack (EN285).

Prior to performing the test, run a 134°C cycle with the chamber empty. Immediately after this cycle place the Daily Control DCG2 in the chamber as closely as possible to the position which during validation is identified as the cold spot.

Carry out a steam penetration test (134°C, max. 3.5 min.) in an otherwise empty chamber.

Upon completion of the cycle remove the Daily Control DCG2 from the sterilizer, open it and take out the indicator sheet.

The sheet must have changed from light green to a uniform dark colour. If any difference in this colour is noted, repeat the test. If this second test also fails, it should be recorded

as a fail on the test sheet.

Record the results on the test sheet with other relevant data. If the test fails, the sterilizer is not functioning properly and needs maintenance, adjustment and/or subsequent validation.

The endpoint reaction is permanent, making the sheet a reliable record when kept on file.

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